



YOU CAN DO IT REMOTELY*



YOU CAN DO IT


Easily share your glucose readings with your
healthcare provider using the FreeStyle LibreLink app


FreeStyle
LibreLink


life. to the fullest.®
Abbott

The FreeStyle Libre flash glucose monitoring system is indicated for measuring interstitial fluid glucose levels in adults aged 18 years and older with diabetes mellitus. Always read and follow the label/insert. The FreeStyle LibreLink app and the FreeStyle Libre reader have similar but not identical features. A finger prick test using a blood glucose meter is required during times of rapidly changing glucose levels when interstitial fluid glucose levels may not accurately reflect blood glucose levels or if hypoglycemia or impending hypoglycemia is reported by the FreeStyle LibreLink app or when symptoms do not match the app readings. The FreeStyle Libre sensor communicates with the FreeStyle Libre reader that started it or the FreeStyle LibreLink app that started it. A sensor started by the FreeStyle Libre reader will also communicate with the FreeStyle LibreLink app. The FreeStyle LibreLink app is only compatible with certain mobile devices and operating systems. Please check the website for more information about device compatibility before using the app. Use of FreeStyle LibreLink requires registration with LibreView.

Easily share your glucose data with your healthcare provider

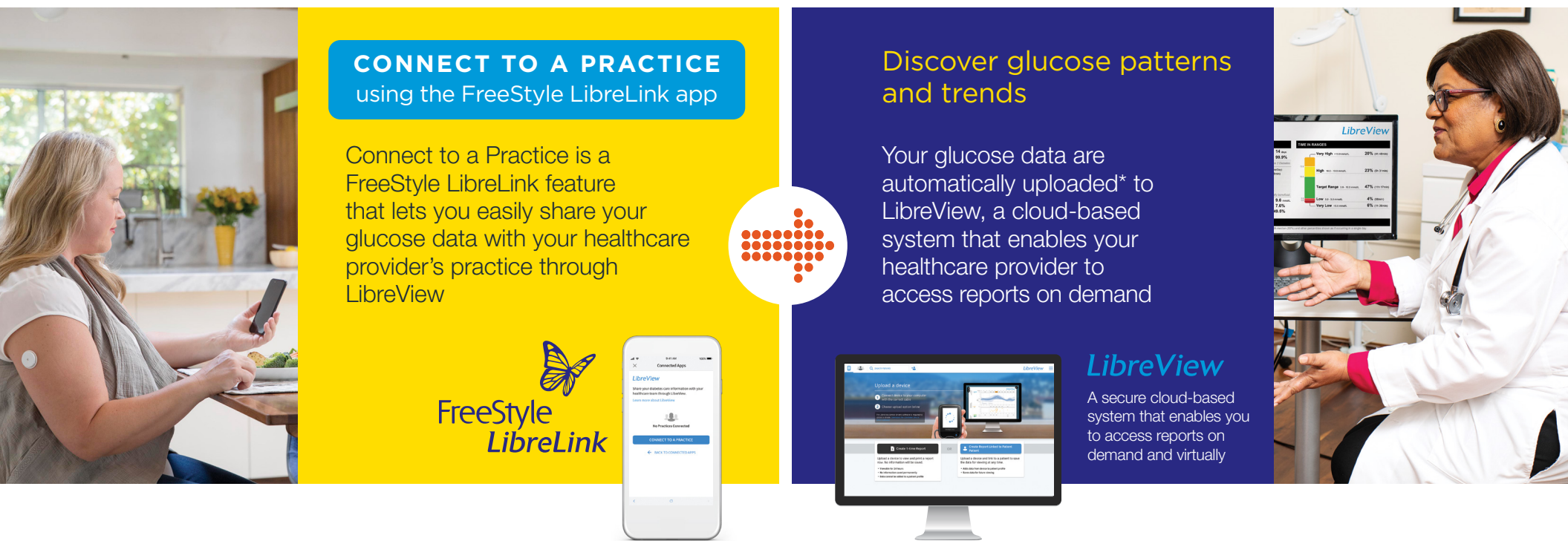

CONNECT TO A PRACTICE
using the FreeStyle LibreLink app

Connect to a Practice is a FreeStyle LibreLink feature that lets you easily share your glucose data with your healthcare provider's practice through LibreView

Discover glucose patterns and trends

Your glucose data are automatically uploaded* to LibreView, a cloud-based system that enables your healthcare provider to access reports on demand

LibreView
A secure cloud-based system that enables you to access reports on demand and virtually

FreeStyle LibreLink

Easily connect

Easy to set up a connection to your healthcare provider's practice through the FreeStyle LibreLink app

Automatic data sharing

Once connected, your glucose data are automatically shared* to your healthcare provider's LibreView account

More productive conversations


Your healthcare provider can access glucose reports on demand to support more productive conversations

For illustrative purposes only. Not actual patient data.

LibreView is developed, distributed, and supported by Newyu, Inc. The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.

* Automatic upload requires a wireless internet connection or mobile data connection.

Easy to start sharing your glucose data using your FreeStyle LibreLink app

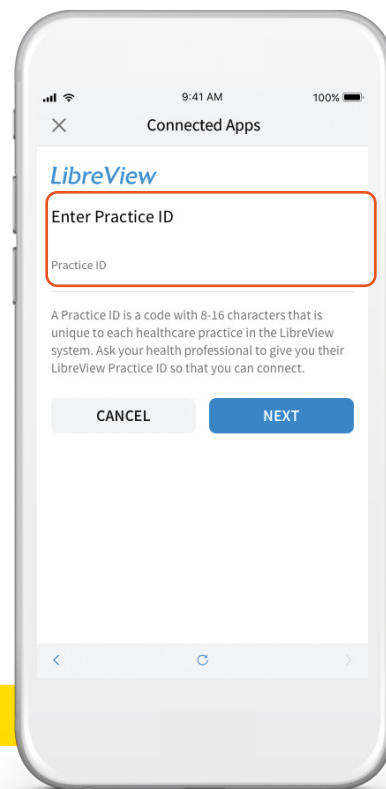

Option 1: You connect to your healthcare provider

Ask your healthcare provider or office staff to provide their LibreView Practice ID, which is a code specific to your healthcare provider's practice


Then simply enter the ID in the FreeStyle LibreLink app

Go to:

*Menu > Connected Apps > LibreView >
Connect to a Practice > Enter Practice ID*

Click **Next** and then **Connect** after viewing your healthcare provider's practice contact information

Your healthcare provider sends you an invitation to share your glucose

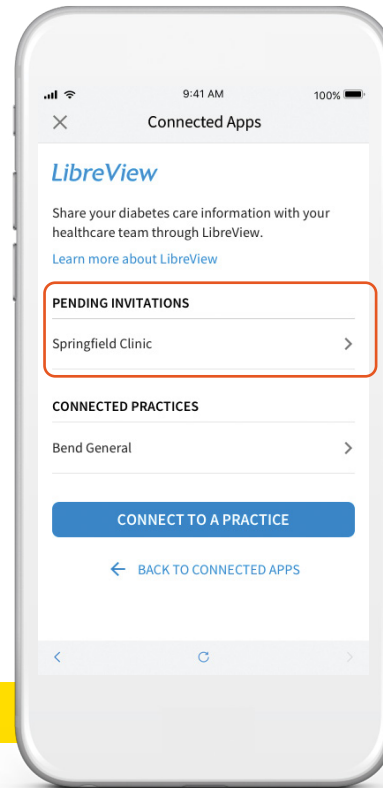

Option 2: Accept practice invitation

Your healthcare provider's office will send an invitation to share your glucose data that can be accepted in two ways:

1. Click on "Accept Invitation" in the email
2. Accept the invitation directly in the FreeStyle LibreLink app

Go to:

Menu > Connected Apps > LibreView > Pending Invitations > Accept

Easily share your glucose data with your healthcare provider for more productive conversations




FreeStyle
LibreLink

CONNECT TO A PRACTICE
Use the FreeStyle LibreLink app to share glucose data with your healthcare provider



MORE PRODUCTIVE CONVERSATIONS

You and your healthcare provider can have more productive conversations about your glucose patterns and trends

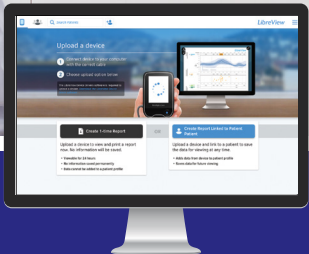

LibreView

A secure cloud-based system that enables you to access reports on demand and virtually



Download the FreeStyle LibreLink app today and enable data sharing



life. to the fullest.®

Abbott