


You can share readings directly with your healthcare provider through LibreView*



Use the **FreeStyle Libre 2 app**[†] to easily share your glucose data with your healthcare provider.

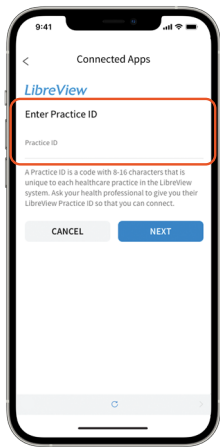

Option 1: Connect with your healthcare provider


Ask your healthcare provider or office staff to provide their LibreView Practice ID, which is a code specific to your healthcare provider's practice.

Then simply enter the ID in the FreeStyle Libre 2 app.

Go to:

Menu > Connected Apps > LibreView > Connect to a Practice > Enter Practice ID

Click **Next** and then **Connect** after viewing your healthcare provider's practice contact information.

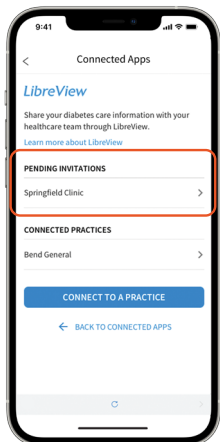

Option 2: Accept practice invitation

Your healthcare provider's office will send an invitation to share your glucose data that can be accepted in two ways:

1. Click **Accept Invitation** in the email
2. Accept the invitation directly in the FreeStyle Libre 2 app

Go to:

Menu > Connected Apps > LibreView > Pending Invitations > Accept



Need help? Contact Abbott Customer Care at **1-888-205-8296** or visit **MyFreeStyle.ca** for more information and helpful videos.



life. to the fullest.®

Abbott

The FreeStyle Libre 2 flash glucose monitoring system is indicated for measuring interstitial fluid glucose levels in people aged 4 years and older with diabetes mellitus. Always read and follow the label/insert.

* The LibreView website is only compatible with certain operating systems and browsers. Please check www.LibreView.com for additional information.

† The FreeStyle Libre 2 app and the FreeStyle Libre 2 reader have similar but not identical features. Finger pricks are required if readings do not match symptoms or expectations. The FreeStyle Libre 2 sensor communicates with the FreeStyle Libre 2 reader that started it or the FreeStyle Libre 2 app that started it. The FreeStyle Libre 2 app is only compatible with certain mobile devices and operating systems. Please check the website for more information about device compatibility before using the app. Use of FreeStyle Libre 2 requires registration with LibreView.

You can also share glucose data from the **FreeStyle Libre 2 reader** with your healthcare provider.



If you scan with the FreeStyle Libre 2 reader, you need to create a LibreView account **BEFORE** you can share your data:

- Go to **www.LibreView.com**
- Click on **Sign Up**
- Choose the **LibreView Account for Patients** and follow on-screen instructions
- **Connect** your FreeStyle Libre 2 reader
- Press **Upload a device** and follow on-screen instructions

There are two easy ways to share data from your reader with your healthcare provider.

Option 1: Connect with your healthcare provider



Ask your healthcare provider or office staff to provide their LibreView Practice ID.
Practice ID: _____

Log in to your LibreView account. Click on the  icon at the top right of the screen and select **Account Settings**.

Then click on **My Practices** and enter your healthcare provider's Practice ID.

Option 2: Accept practice invitation

Ask your healthcare provider to invite you to share your data. You will receive an email inviting you to share.

Click **Accept Invitation** in the email and you'll be directed to the LibreView website, where you can log in to your LibreView account.



Need help? Contact Abbott Customer Care at **1-888-205-8296** or visit **MyFreeStyle.ca** for more information and helpful videos.



The FreeStyle Libre 2 app and the FreeStyle Libre 2 reader have similar but not identical features. Finger pricks are required if readings do not match symptoms or expectations. The FreeStyle Libre 2 sensor communicates with the FreeStyle Libre 2 reader that started it or the FreeStyle Libre 2 app that started it. The FreeStyle Libre 2 app is only compatible with certain mobile devices and operating systems. Please check the website for more information about device compatibility before using the app. Use of FreeStyle Libre 2 requires registration with LibreView.

© 2022 Abbott. FreeStyle, Libre, and related brand marks are marks of Abbott. Product images are for illustrative purposes only. Other trademarks are the property of their respective owners.
ADC-40819 v2.0